PRESS *key*

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DIVAMED Press Key **Your smart business partner**

With DIVAMED Press Key you can offer to your customers a complete and customized program for the skin blemishes treatment caused by lymphatic stasis. Very easy and quick to use DIVAMED Press Key will be in service of your beauty centre.

focus: What is Pressotherapy?

Pressotherapy for the treatment of lymphoedema has been proposed since 1950. The effect is based on a physical action that acts on the venous and lymphatic circulation making it more efficient and consequently improving the more frequent blemishes. The pressure is not exercised at the same time on all body surface covered by the leggings, but according to the centripetal sequence followed by the venous blood and the lymph, i.e. from feet to pelvis. The interstitial cells' fluid and his solutes are drained by balanced pressure to the tissues. Therefore blood circulation is stimulated and the extracellular environment is free from waste.

Pressotherapy actions

- Lymphodrainage;
- Improving of the skin tone;
- Treatment of Cellulite to different stages;
- Metabolism reactivation;
- Improving of the venous- lymphatic circulation;
- Treatment of Tired- legs syndrome;
- Treatment of Build- up of localised fat;
- Treatment of Oedemas localised and spread.

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DIVAMED Press Key

Innovation for your beauty centre

The new **Pressotherapy DIVAMED Press Key with 8 independent sectors** is the fusion between available medical science knowledge and the most advanced electronic and mechanical technologies.

A wide selection of programs ①

DIVAMED Press Key is provided by **51 developed programs** controlled by a ARM 32 bit microprocessor able to manage the revolutionary algorithm elaborated by Eurtronik, based on problems to treat, changing in real time the 8 sectors' pressure and the frequency according to the medical dictates.

Customized diagnosis ②

The careful **customized diagnosis** finalized for blemishes treatments referring to venous and lymphatic system, is the result of the analysis that Eurtronik's algorithm takes out from the comparison with the personal data of the client, like his physical, physiological, pathological and blood circulation characteristics. Thanks to Eurtronik's analytic algorithm and starting from its 55 base programs, DIVAMED Press Key develops countless programs, made for each person to care.

Faster set up and free updates by USB pen ③

After the loading of all the patient's data, these data are saved on an **USB PEN** able to contains 500 clients. . That archive represents the entire treatments cycle central core. The device shows automatically the progress report resulting from the treatments already done and the ones not yet done permitting a very quick control of situation.

Moreover every time Eurtronik releases new programs USB pen let to update DIVAMED Press Key's firmware without technical service. **The update is free.**

Intuitive interface ④

With DIVAMED Press Key you can make available for your staff **easy and fast device to manage, thanks to an intuitive interface studied to drive you in the treatments set up.**

DIVAMED Press Key automatically generates customized diagnosis, but it gives also the possibility possibility to select a single treatment, basically on his own experience, but with Eurtronik's algorithm assistance.

More security for your customers ⑤-⑥

DIVAMED Press Key is provided with an **electronic Sphygmomanometer** checking arterial blood pressure and performing safe and effective treatments, compliant with scientific criteria.

DIVAMED Press Key is also provided with a **safety button**, to guarantee the maximum safety for the client during the treatment, without the user intervention.

Privacy guarantee

DIVAMED Press Key is programmed **to respect the customers privacy:** Instead of inserting personal details only encrypt data alphanumeric code are to be key in.

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DIVAMED Press Key KIT POINT

*Kit Point with 8 **INDEPENDENT SECTORS** is formed by Ankle-boots, Leggings, Abdomen band, Buttocks and hips band with also the inguinal air bands to avoid liquid accumulation by stimulating relative lymph nodes.*

Kit point materials, made in Italy quality

Is used Nylon 210/D, employed for the hospital incubators. They are prewashed, hypoallergenic, antifungal and aseptic. Moreover the internal covering is made in aseptic, anti bacterial, antifungal polyurethane M3, responding to the FR specifics. The air bladders are in polyurethane PU39 welded with high frequency with antifungal and antibacterial treatment.

The bag has been studied with apposite pockets to contain and to shelve all the constitutive elements. Once folded it is placed in the handy case supplied.

DIVAMED Press Key FEATURES More controls

Every air bladder pressure in electronically controlling by a sensor (for a total of 9 sensors) that, together with measuring the preset level for each sector, allows to change during treatments without stopping it. It also let to report to user the damaged air bladders.

A longer pumping tubes duration

The PUMPING TUBES of the air bladders has been especially designed and constructed with a special plastic material to eliminate the "memory effect" that is, if they are folded up, their form does not change and return to the initial position besides.

Incredibly silent

The air is fed by a **compressor** properly silenced and cushioned to ensure an high comfort during treatment. The flow of air to the inflation sectors is realized by two sets of **rotating valves** as an alternative to traditional solenoid notoriously noisy.

Nice to see, handy to use

The connection between the device and kit point is made from a specially constructed **connector** which includes rationally every connection tubes. It also is provided with a sensor that alerts the operator if the device connection has not been completed correctly.

Improved air bladders' hygiene and preservation

*The AIR that arrives to the device in aspiration from the external is **CLEANED UP** from impurities and micro-dusts by a specific aspiration filter while, the one send to the air bladders is dehumidified by an **anti-humidity filter**, placed on the back of the device.*

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DIVAMED Press Key Technical Specification

The device consists of:

- 1 DIVAMED Press Key device with **monochrome display and keyboard controls**;
- 1 case with Kit Point;
- 1 USB pen drive;
- 1 Sphygmomanometer;
- 1 Safety button and 1 Cable of supply.
- 1 User manual

Optional:

- TNT trousers (10 pcs.);
- Cartene bootlegs (10 pcs.);
- Cosmetic cream for pre-post treatment;
- Kit Point for the treatment of arms.

Weight DIVAMED: 8 Kg, Dim. : L380 x P450 x H 250 mm;
Weight case: 4 Kg, Dimensions: L580 x P490 x H 150 mm

The device respects the actual laws related to the two directives aimed at the **CE** signature: 2004/108/CE: electromagnetic compatibility (EMC); 2006/95/CE : low tension.

Available **CE Mark MEDICAL device (2007/47/CEE)**.

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